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A Power Switch  
B Low-Voltage Warning  
C Oxygen Saturation Level  
D Pulse Rate Bargraph  
E Pulse Rate

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GB INSTRUCTIONS – PULSE OXIMETER

Intended Use

For non-medical periodic check of heart rate and blood oxygen saturation level at home, and for sport and recreational uses such as mountain climbing, high altitude activities and running.  
This fingertip pulse oximeter is not intended for medical diagnostic or treatment use.

Cautions

- Do not use if in cardiac arrest or in shock.
- The Oximeter has no alarms and is not intended for continuous monitoring.
- SpO<sub>2</sub> measurements may be adversely affected in the presence of strong light. Shield the sensor area with a cloth if necessary.
- Fingernail polish or false fingernails may cause inaccurate SpO<sub>2</sub> readings.
- Inaccurate measurements may be caused by immersing the sensors in liquid.
- Excessive movement may cause inaccurate readings.
- Keep away from small children as this device has small parts that may pose as a choking hazard. Follow local ordinances for disposal and recycling of the device, components and batteries.

Properties

- Fingertip Oximeter is a portable non-invasive, spot-check for blood oxygen saturation (SpO<sub>2</sub>) and pulse rate.
- The Oximeter requires no routine calibration other than replacement of batteries.
- Operation of the product is simple and convenient.
- The product is small in volume, light in weight and convenient to carry.
- Power consumption of the product is low and two AAA batteries is enough for it to operate continuously for 24 hours.
- A low voltage warning will be indicated on the LED display when battery power is low, replace batteries immediately to avoid inaccurate measurements.
- The product will automatically be powered off when no finger is inserted into the product within 8 seconds.

Installation Battery

- Slide the battery cover and remove.
- Insert two AAA batteries with correctly aligned polarities.
- Replace the battery cover.
  - Battery polarities should be correctly aligned. Otherwise, you may damage the device.
  - Please remove the batteries if the Oximeter will not be used for a long time.

Installation Lanyard

- Thread the thin end of the lanyard through the loop at the top of the Oximeter.
- Thread the thick end of the lanyard through the thin end, and then tighten to secure.

Instructions

- Clamp one finger fully into the rubber hole of the Oximeter.
- Press the power switch on the front panel.
- Sit down and relax your whole body while the measurement is being taken.
- Read corresponding data on the display screen.

When your finger is inserted into the Oximeter, your nail must be facing upwards.

The pulse rate bar graph displays your pulse beat. The height of the bar graph shows the strength of your pulse.

Maintenance & Storage

- Replace the batteries immediately when the low voltage warning lights up.

- Clean the surface and the medical rubber of the oximeter before usage.
- Remove the batteries inside the battery compartment if the oximeter will not be operated for a long time.
- It is best to store the product in a place where temperature is between –40°C~60°C (–40°F~140°F).
- It is recommended that the product should be kept in a dry environment at all times. A damp environment may decrease its durability or even may damage the product.

Product Accessories

- Lanyard
- 2 x AAA Batteries (not included)
- Instruction Sheet

Specifications

Display: LED Display;  
SpO<sub>2</sub> Measuring Range: 0% – 100%;  
Pulse Rate Measuring Range: 30 bpm – 250 bpm;  
Pulse Intensity Display: Columniation display  
Power Requirements: 2 x1.5V AAA alkaline battery (or rechargeable battery).  
Resolution: 1% for SpO<sub>2</sub> and 1 bpm for Pulse Rate.  
Measurement Accuracy: ±2% in stage of 70%–100% SpO<sub>2</sub>, and meaningless when stage being smaller than 70%. ±2 bpm or ±2% (select larger) for Pulse Rate.  
Measurement Performance in Weak Filling Condition: SpO<sub>2</sub> and pulse rate can be shown correctly when pulse-filling ratio is 0.4%. SpO<sub>2</sub> error is ±4%, pulse rate error is ±2 bpm or ±2% (select larger).

Resistance to surrounding light: The deviation between the value measured in the condition of man-made light or indoor natural light and that of a dimly lit room is less than ±1%. It is equipped with a function switch. The Oximeter will power off if no finger is inserted into the Oximeter within 8 seconds.

Optical Sensor

Red light (wavelength is 660nm, 6.65mW)  
Infrared (wavelength is 940nm, 6.75mW)

Problem	Possible Reason	Solution
The SpO <sub>2</sub> and Pulse Rate can not be displayed normally	1. The finger is not properly positioned. 2. Your SpO <sub>2</sub> is too low to be detected.	1. Place the finger properly and try again. 2. Try again; Go to a hospital for a diagnosis if you are sure the device works all right.
	1. The finger is not placed inside deep enough. 2. The finger is shaking or the you are moving.	1. Place the finger properly and try again. 2. Keep the user calm and at ease.
The device can not be turned on	1. The batteries are drained or almost drained. 2. The batteries are not inserted properly. 3. The device is malfunctioning.	1. Change batteries. 2. Reinstall batteries. 3. Please contact the local service center.
	1. The device will power off automatically when it gets no signal within 5 seconds. 2. The batteries are almost drained.	1. Normal. 2. Change batteries.

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