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**F** INSTRUCTIONS

**I** ISTRUZIONI

**GB** INSTRUCTIONS

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22515 | 022016 | Made in China

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### **Einführung**

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**Introduction**

- Thank you for purchasing the Fully Automatic Wrist Style Digital Blood Pressure Monitor
- The unit uses the oscillometric method of blood pressure measurement. It means the unit detects the movement of your blood through your brachial artery, and converts your blood pressure into a digital reading. The unit is simple to use because a stethoscope is not needed while using an oscillometric monitor.
- The unit stores automatically 120 sets of measurement values. You can read the stored data conveniently by pressing the memory buttons.

**Important Safety Notes**

To assure the correct use of the product basic safety measures should always be followed including the precautions listed below:

- Read all informations in the instruction manual and any other literature included in the box before using the unit.

<b>Attention sign</b>	
	Attention sign
	Zwingend
	Verboten
	Type B applied part
	Consult accompany documents
	Seriennummer
	Symbol for the marking of electrical and electronics devices according to Directive 2002/96/EC. The device, accessories and the packaging have to be disposed of waste correctly at the end of the usage. Please follow Local Ordinances or Regulations for disposal.

- Contact your physician for specific information about your blood pressure. Follow the instructions of your healthcare provider.
- Operate the unit only as intended. Don't use for

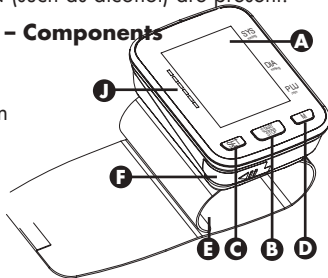
any other purpose.

- The unit is intended to use on measuring blood pressure and pulse rate for adult only, not recommended for neonatal baby.
- Do not use a cellular phone near the unit. It may result in operational failure.
- Please avoid using in high radiant area in order to make your measuring data correctly.
- Do not disassemble or attempt to repair the unit or components.
- Do not use the equipment where flammable gas (such as anesthetic gas, oxygen or hydrogen) or flammable liquid (such as alcohol) are present.

### **Know your unit – Components**

#### *Body & Cuff*


- A Display
- B Start/Stop Button
- C SET Button
- D M Button
- E Wrist Cuff
- F Battery Compartment

### Display

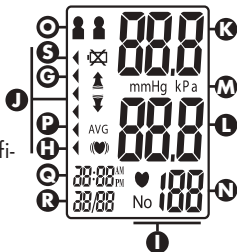- G Inflation/Deflation
- Symbol or WHO Indication
- H Irregular Heartbeat Symbol
- I Memory & Times
- J WHO Blood Pressure Classification Indicator
- K Systolic Blood Pressure
- L Diastolic Blood Pressure
- M Unit Indication
- N Pulse Display
- O Groups
- P Last 3 results average
- Q Time
- R Date
- S Low Battery Symbol

### Power

- 2 AAA-Alkaline Batteries (not included)

## Battery Installation/ Replacement

- 1 Remove the battery cover from the battery compartment.
- 2 Insert two AAA powerful batteries into the compartment and ensure each battery is in the proper direction.
- 3 Replace battery cover.

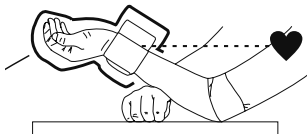
## Caution

- Use two identical 1.5V alkaline batteries type AAA LR03.
- Replace all of the batteries if the symbol of low battery appears on the display.
- Never leave any low battery in the battery compartment since they may leak and cause damage to the unit.


## **Proper way of measurement**

- 1 Remove all clothes from your arm allowing the cuff to fit directly on the skin.
- 2 Wrap the cuff tightly around your wrist by using the Velcro strip.
- 3 Turn your palm upwards, stretch your fingers and wrist naturally.
- 4 Use a soft stuff below the BP monitor to support your arm as the figure shown below.
- 5 Keep your wrist at the same level of your heart.
- 6 Sit upright in a chair, and take about 10 times deep breath to relax your entire body, especially between your elbow and fingers. Then turn on the unit and start to measure.


**No touching**


## Proper way of measurement – Finger's pose

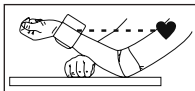Not good

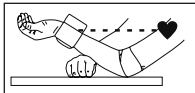
**Good**

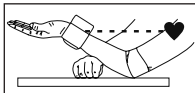
Not good


## Proper way of measurement – Wrist's pose

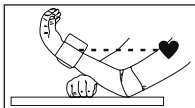Not good

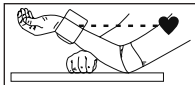
**Good**

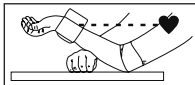
Not good

## How to take a measurement

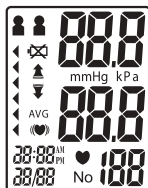
- 1 Press the START/STOP button.  
All display symbols appear on the screen. The cuff starts to inflate automatically.

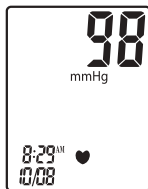- 2 Measurement starts while inflating  
The heartbeat symbol flashes once a pulse is detected.

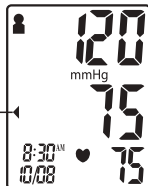- 3 The blood pressure and pulse rate are displayed when the measurement is completed. The cuff is deflated automatically, and the measurement results are stored in the memory. The (♥) symbol will be displayed if arrhythmia is detected.

- 4 Press the START/STOP button to turn off the unit.  
The unit will automatically turn off after two minutes if it is not used in that time. Please rest for 30–60 seconds before taking your next measurement.


### **How to use the memory function**

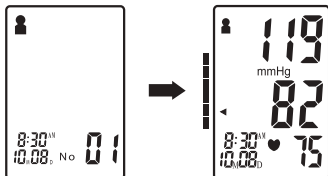- The unit stores the blood pressure and pulse rate in the memory after completing each measurement. It can automatically store 120 sets of measurement values. The earliest record is deleted to save the latest measurement value when there are more than 120 sets.
- The unit also calculates an average (AGV) reading these are based on the last 3 measurements taken.
- To check the average results from other groups, select the desired group first prior to activating the «M»-button in the off position.

## To read the memory values

- Press the «M» button to view the last measurement value.

## To clean out the memory values

When reading the measurement values, press and hold the Ⓜ memory button, then press START/STOP button for 3 – 4 seconds.

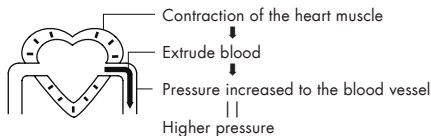
## About Blood Pressure/Blood Circulation

The blood circulation is responsible for supplying the body with oxygen. Blood pressure is the pressure exerted on the arteries.


The systolic blood pressure value (higher pressure or top value) represents the blood pressure produced by contraction of the heart muscle.

The diastolic blood pressure value (lower pressure or lower value) represents the blood pressure produced by relaxation of the heart muscle.

## *Systolic Blood Pressure*

## *Diastolic Blood Pressure*


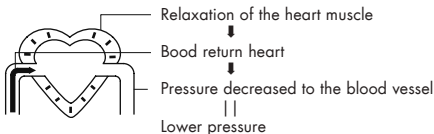
## **Classification of Blood Pressure**

After each measurement is completed, the LCD display will show your position automatically on the six segments of the bar indicator which corresponds to WHO Blood pressure Indicator.


### **\*Note**

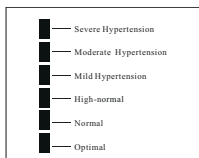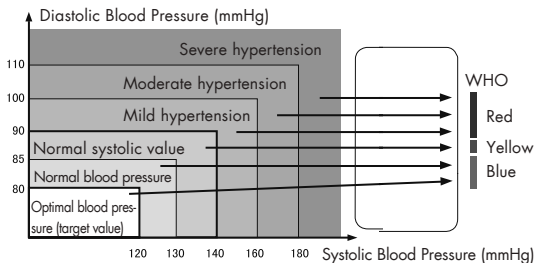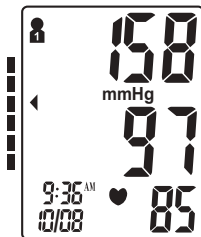When a person's systolic and diastolic pressures fall into different categories, the higher category should apply.

Mild Hypertension

Blood Pressure Classification Indicator

## Error Indicators

The following symbol will appear on the display when measuring abnormal.

Symbol	Cause	Correction
	The course of inflating appears error.	<ol style="list-style-type: none"><li>1. Wrap the cuff correctly and tightly.</li><li>2. Inflate over again after ensuring.</li></ol>
	When measurement fails.	<ol style="list-style-type: none"><li>1. Do not move your arm and body and keep quiet.</li><li>2. Measure over again according to correct way.</li></ol>
	Pressurization is over 300mmHg.	Read user manual carefully and re-test properly.
	When the batteries power are too low.	Replace all of the worn batteries with two new ones.

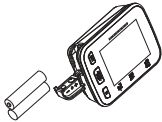
## Eliminate Faults

Problem	Causes and Solutions
No power.	Replace all the worn batteries with two new ones.
No display appears on the display screen.	Check the battery installation for proper placement of the batteries polarities.
Measurement values appear too high or too low.	Blood pressure varies constantly. Many factors including stress, time of day, how you wrap the cuff, may affect your blood pressure. Review the sections «Proper Way of Measurement» and «Take a Measurement».

## Care

- Keep the unit in the storage case when not in use.
- Clean the unit with a soft dry cloth. Do not use any abrasive or volatile cleaners.
- Never immerse the unit or any components in water.
- Use the unit consistent with the instructions provided in this manual. Use only authorized parts and accessories.

## Unterhalt

<p>Do not clean the body and cuff with naphtha, thinner or gasoline etc.</p>	
<p>Store the unit in a clean and dry location. Do not subject the unit to extreme hot or cold temperature, humidity and direct sunlight.</p>	
<p>Do not wet the cuff or attempt to clean the cuff with water.</p>	
<p>Remove the batteries if the unit will not be used for 3 months or longer. Always replace all of the batteries with new ones at the same time.</p>	

## Specifications

Model No.		BSP-22
Display		LCD Digital Display
Measuring Principle		Oscillometric Method
Measurement Localization		Wrist
Cuff		Soft Wrist Cuff
Measurement Range	Pressure	0–300 mmHg (0–39.9 kPa)
	Pulse	30–180 Pulses/min
Accuracy	Pressure	$\pm 3$ mmHg (0.4 kPa)
	Pulse	$\pm 5\%$ of reading
LCD Indication	Pressure	3 Digits Display of mmHg or kPa
	Pulse	3 Digits Display
	Symbol	Memory/IHB/Heartbeat/Low Battery, etc.
Inflation		Automatic by Internal Pump
Rapid Air Release		Automatic by Air Valve

<b>Specifications</b>		
Memory Function		Memory 120 Sets Measurement Values
Power Source		2 AAA Alkaline Batteries, D.C.3V
Automatic Power-Off		In 3 Minutes
Gross Weight		Approx. 100g (Batteries not included)
Operating Environment	Temperature	10~40°C (50 ~ 104°F)
	Humidity	15~90% RH (Noncondensing)
Storage Environment	Barometric Pressure	105~80 kPa
	Temperature	-20~40°C (-4 ~ 104°F)
	Humidity	10~95% RH (Noncondensing)

## Statement

- The intended use: The unit is intended to be used by adults at home or medical centre to measure blood pressure and pulse rate.
- The unit satisfies the requirements of  
EN60601-1 Medical electrical equipment 1,  
EN60601-1-2 Medical electrical equipment 2,  
EN1060-1: Non-invasive sphygmomanometer,

EN1060-3: Non-invasive sphygmomanometer,  
EN1060-4: Non-invasive sphygmomanometer.

- The unit has passed the EMC Test.
- Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultatory method, within the limits prescribed by the American National Standard, Manual, electronic, or automated sphygmomanometers.
- The risk of patient and user can be lowered to acceptable level.
- Substitution of a cuff different from that supplied might result in measurement error.
- Specifications are subject to change without notice due to improvements in performance.

UBTC Switzerland AG    Importer EU: UBTC Germany GmbH  
Blegistrasse 19    Alte Provinzialstrasse 21  
6340 Baar – Switzerland    58708 Menden – Germany

[www.UBTC.biz/Daily](http://www.UBTC.biz/Daily)

22515 | 032016 | Made in China